


# JĒBA Measurement Guide

**PLACE YOUR ORDER TODAY!**

Phone 800.551.3008 | Fax the order form 877.551.3001

Email the order form to: [custserv@theorthoticgroup.com](mailto:custserv@theorthoticgroup.com)

## Measuring Guide

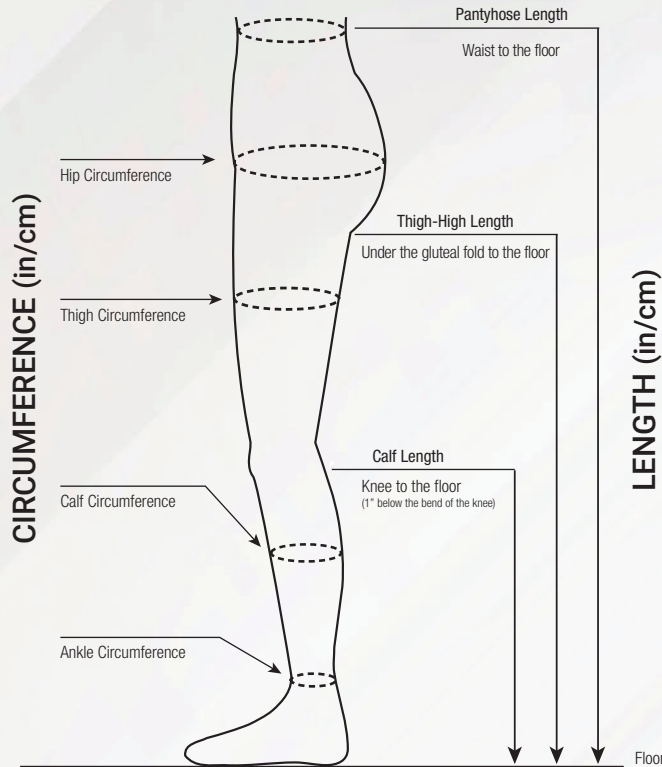

### MEASUREMENT GUIDELINES (inches/centimeters)

#### CIRCUMFERENCE

##### Hip (Center of Hips)

Measure around the widest part of the hips

##### Thigh (Largest Thigh)

Measure around the widest part of the thigh (right & left side)

##### Calf (Largest Calf)

Measure around the widest part of the calf (right & left side)

##### Ankle (Smallest Ankle)

Measure just above the ankle bone (right & left side)

#### LENGTH

##### Pantyhose (Waist Length)

Waist at the belly button to the floor

##### Thigh-High (Thigh Length)

Gluteal fold to the floor

##### Knee-High (Calf Length)

One inch below the bend at the knee to the floor

## Sizing Chart

	Smallest Ankle	Largest Calf	Calf Length	Largest Thigh	Thigh Length	Center of Hips	Waist Length
Small	7" - 8¼" / 18 - 21cm	11" - 15" / 28 - 38cm	to 15½" / to 39cm	15¾" - 24¾" / 40 - 62cm	23" - 26" / 58 - 66cm	28" - 45" / 71 - 115cm	29" - 35" / 74 - 89cm
Medium	8¾" - 9¾" / 21 - 25cm	11¾" - 16½" / 30 - 42cm	to 17" / to 43cm	18½" - 27½" / 46 - 70cm	23" - 29" / 58 - 74cm	30" - 50" / 76 - 127cm	31" - 37" / 79 - 94cm
Large	10" - 11¾" / 25 - 29cm	12½" - 18¾" / 32 - 46cm	to 19" / to 49cm	21½" - 30¾" / 54 - 78cm	25" - 31" / 64 - 79cm	33" - 54" / 83 - 137cm	33" - 39" / 84 - 99cm
X-Large	11½" - 13" / 29 - 33cm	13¾" - 19¾" / 34 - 50cm	to 20" / to 51cm	23¾" - 32" / 60 - 81cm	27" - 33" / 69 - 84cm	40" - 65" / 102 - 166cm	35" - 41" / 89 - 104cm

### RETURN POLICY

We offer a no-restocking-fee policy to return Jēba within 30 days of purchase, provided the Jēba™ compression hosiery has not been worn and is returned in the original, undamaged packaging. A credit will be issued on your next accounting statement.

The following is required for all returns: Completed Return Form with the original invoice number, full description of the merchandise, reason for the return, and product packaged in its original undamaged package. Damaged packaging and worn product will not be restocked and will be returned to the sender. Acceptance of all returned merchandise is at the discretion of Apex Foot Health Industries.

Prices Effective January 1, 2020